



Grommet Top Strap

INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. SH025-2

1. Adjust each shoulder strap for a comfortable fit, according to the user's shoulder width and sitting position. Mark for mounting screws to hold this position.
2. Mounting straps can be attached directly to solid back surfaces. Use 6mm (1/4") Flat head or Button head screws to mount to chair frame or solid back.

Mounting screws must have a minimum pullout strength of 90kg (200lbs). (Not included in package.)

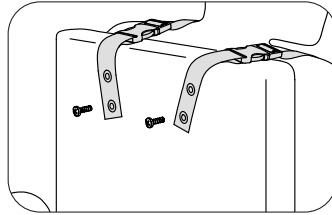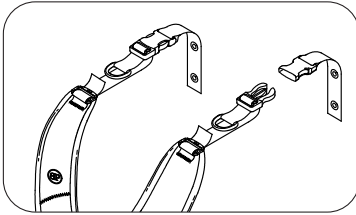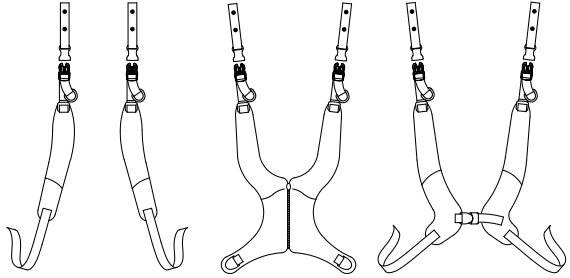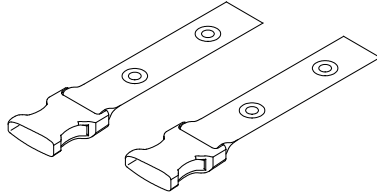MAINTENANCE: Check periodically for signs of wear in the stitching, webbing, and pads. If significant wear is found, contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons—health and safety depend on it!

CLEANING: Machine wash, hot, 60°C (140°F) using laundry detergent. Tumble dry, or drip dry. Do not iron.

WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA, go to www.bodypoint.com

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